Optelec *ClearView* +

User Manual
Bedienungsanleitung
Manuel d'utilisation
Gebruikershandleiding
Manuale utente
Manual del usuario
取扱説明書
使用手冊

Optelec

ClearView +

User Manual	English
Bedienungsanleitung	Deutsch
Manuel d'utilisation	Français
Gebruikershandleiding	Nederlands
Manuale utente	Italiano
Manual del usuario	Español
取扱説明書	日本語
使用手冊	繁體中文

Version 7.1.3

Copyright 2005 Optelec, the Netherlands

All rights reserved

Optelec
P.O. Box 399
2993 LT Barendrecht
The Netherlands

Telephone: +31 (0)88 6783 444. Fax +31 (0)88 6783 400

E-mail: info@optelec.nl

Website: www.optelec.com

ClearView⁺ User Manual

Optelec, part of Tieman Group, was founded in 1975 in Rockanje, the Netherlands. Since 1975, Optelec has been developing and producing innovative products to assist vision-impaired people with their daily activities to maintain and regain independence.

Visit Optelec's website at www.optelec.com for more information on products and services.

Every effort has been made to ensure that the information contained in this user manual is correct at the time of publication. Neither Optelec, nor any of its associates, shall be held responsible for any errors or damages that may result from any incorrect use of the information contained in this manual.

Index

1.	Introduction.....	5
2.	Safety and Care.....	7
3.	Unpacking, Installation and Setup	9
4.	Operation.....	11
	Operation – Switching ON / OFF.....	11
	Adjusting the magnification	12
	Choosing viewing modes	11
	Auto-focus lock.....	12
	Object lighting ON /OFF	13
	Adjusting image control	13
	Adjusting the brightness	13
	Displaying system information on screen	13
	Locking the reading table	14
	Using the footswitch	14
5.	Feature Packs	15
	5.1 Feature Pack Basic.....	15
	Semi-colors	15
	Find function (position locator)	16
	Pointer (position locator)	16
	5.2 Feature Pack Advanced	17
	Switching between the ClearView ⁺ and a computer / external source.....	17
	Activating line markers / window shades.....	18
	Positioning line markers / window shades.....	18
	Sizing the line markers / window shades.....	18
	Horizontal / vertical button.....	19
	5.3 Zero Button Feature Pack	19
	Switching ON / OFF	19
	Auto Zoom fixed font technology	19
	5.4 Auto Zoom Feature Pack	20
	Auto Zoom ON/OFF	20
	Adjusting the Auto Zoom	20

6.	PC units with the Feature Pack Advanced	21
	Semi-color combinations	21
	Pointer (position locator)	21
	Activating line markers / window shades.....	22
	Switching between the ClearView ⁺ and computer / external source.....	22
	Sizing the line markers / window shades & Picture-in-Picture image	22
	Positioning lines / windows & the Picture-in-Picture image	23
	Horizontal / vertical button.....	23
7.	Adjusting the monitor.....	24
8.	Troubleshooting	25
9.	Contacting Product Support.....	27
10.	Warranty Conditions.....	27
11.	Compliance	28
12.	Specifications	29

1. Introduction

Congratulations on choosing the ClearView⁺, an auto-focus desktop electronic magnification system developed by Optelec in the Netherlands. The ClearView⁺ enables you to read, write, view photos, do work and much more.

The ClearView⁺ magnifies and enhances text, objects and images in different colors, brightness and contrast. It is easy to operate and can be used at home, in school or at work.

The ClearView⁺ has a unique ergonomic design that focuses on simplicity. The quality reading table allows users to move text and objects smoothly beneath the camera and it integrates the user-friendly control panel. The auto-focus feature has been incorporated as standard in every ClearView⁺ model. Customized ClearView⁺ monitors can be adjusted to prevent glare and light reflections as well as to ensure a more comfortable reading posture.

This manual will help you become familiar with the ClearView⁺ features and operation. Please read this manual thoroughly before setting up and use.

Please note that the ClearView⁺ is a modular system. Different ClearView⁺ models can be assembled by selecting the various modules. This user manual covers all of the models currently available. A number of features and functions can be added by installing different modules. However it may be possible that your model does not support some of the functions or features described in this manual.

If you have any questions or suggestions concerning the use of this product, please contact your Optelec distributor or Optelec's headquarters by consulting the contact information page found at the end of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your ClearView⁺.

2. Safety and Care

Please read the instructions carefully before setting up your ClearView⁺. Keep this user manual for future reference.

- Use the ClearView⁺ (CV⁺) indoors. Do not expose the CV⁺ to extreme or rapid changes in temperature or humidity. Do not place the ClearView⁺ near a radiator or heater, or where it may be exposed to direct sunlight. These conditions may damage the CV⁺'s electronic or optical components.
- Do not place the CV⁺ on an unstable surface. It may fall, causing serious damage to the unit, or injury to the user.
- If using your own monitor mounted on top of the ClearView⁺ system, make sure it does not weigh more than 16 kg, and that it is not larger than a 20" screen. The ClearView⁺ Monitor Platform was designed to support a maximum of 16 kg.
- Firmly grip the monitor by the surrounding frame when positioning to ensure stability to the entire system. When readjusting the 19" or the 22" monitor with the five-directional ultra-flexible TFT arm, please ensure the monitor does not tilt or rotate against the wings of the electronic box.
- The reading table and the monitor arm contain movable parts. Please be careful when adjusting the screen position, or when operating the reading table as to prevent fingers or any loose clothing from getting caught in the operating mechanism. If you have difficulty operating the CV⁺ safely, please seek assistance or contact your Optelec distributor.
- Contact your Optelec distributor to service this device if necessary. Do not open your CV⁺; this will void the warranty.

- When moving the ClearView⁺ to another location, remove the monitor from the base unit for safe and easy transportation. Carry the ClearView⁺ base unit by using the two grips located on either side of the base unit table mid-way between the front and back of the unit at table level.
- Always turn the power off when the CV⁺ is not in use.
- Always unplug the CV⁺ before cleaning it. Use a soft, damp cloth to clean the exterior surfaces. Do not use any cleaning agents or any abrasive materials as these will damage your CV⁺.
- Only use the mains power supply provided.

3. Unpacking, Installation and Setup

The ClearView⁺ is a modular system with a standard VGA output that allows you to connect your ClearView⁺ to any VGA-compatible monitor or TV.

Your ClearView⁺ consists of the following:

- **A base unit** with a reading table and integrated control panel;
- **An electronic box** with the camera module, light sources and input / output connectors. You have the choice between:
 - Monochrome (grey scale black and white)
 - Color
 - HD (for superior image quality)
 - PC (to display the ClearView⁺ image, the PC image, or both together in split-screen or Picture-in-Picture)
- **Customized ClearView⁺ monitor.** The ClearView⁺ offers the following monitor options:
 - 17" TFT monitor
 - 18" TFT monitor (widescreen)
 - 19" TFT monitor
 - 22" TFT monitor (widescreen)

For adjusting the different monitors to your preferred position, please refer to chapter 7.

These monitors may only be fitted as an integrated part of the unit.

- **Feature Packs** (optional) for additional ClearView⁺ functionality. Optelec offers four feature packs:

- Feature Pack Basic (with Semi-color, Find and Pointer functions)
- Feature Pack Advanced (with Semi-color, Find and Pointer functions, Line markers, Window shade and PC-switch functions)
- Zero Button Feature Pack (with the unique fixed font technology)
- Auto Zoom Feature Pack (with Semi-colors, Find and Pointer functions and with the unique adjustable fixed font technology)

The Feature Pack Advanced is supplied as standard with the PC unit.

For operating the feature packs, please refer to chapter 5 and 6.

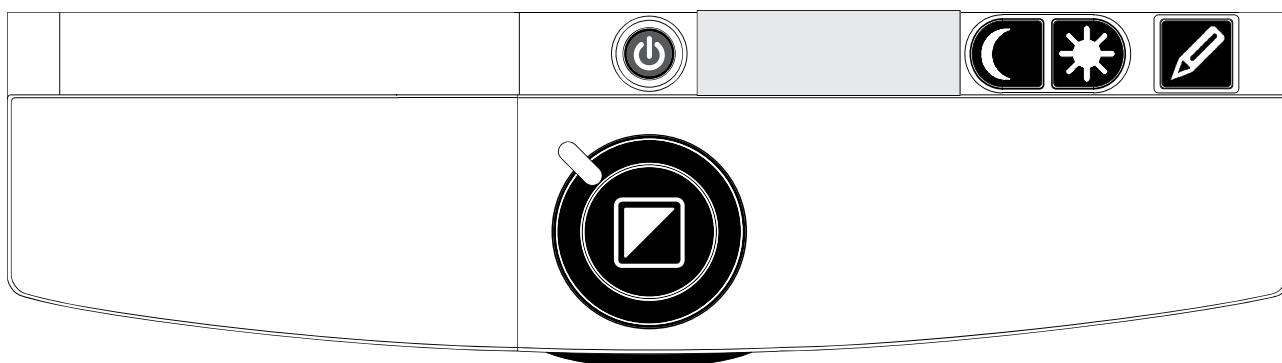
- **Footswitch** (optional) for toggling between the images of the ClearView⁺ and of external sources, such as computers (for Color, HD and PC units).
- **Power supply and cables** for connecting the ClearView⁺ to an electrical wall outlet.
- **This user manual and an installation guide.**

Please note that if your ClearView⁺ system was delivered to you in separate modules, unpack each module carefully before proceeding with installation, and consult the accompanying installation guide to install the different modules. Please contact your Optelec distributor for further instructions if required.

4. Operation

When using the ClearView⁺, place your hands on the wrist support at the front of the table for easier access to the control buttons, and for steady table movement.

If you are using the ClearView⁺ with the Zero Button Feature Pack, please refer to the paragraph "Feature Pack – Zero Button" in chapter 5.

Operation – Switching ON / OFF

To power both the system and monitor, press the ON / OFF button that is located right above the magnification button on the reading table. The ClearView⁺ will start up in the same viewing mode as when it was powered off.

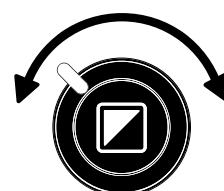
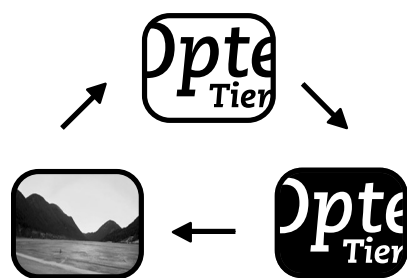
It takes about 10 seconds for the camera unit to initialize before it displays an image. Within about 1 minute, the ClearView⁺ system's light source will achieve full brightness, providing optimal image quality.

Please note that a non-ClearView⁺ monitor must be switched on and off separately.

Adjusting the magnification


Locate the zoom button in the centre of the table, and turn it clockwise to increase the magnification, and counter-clockwise to decrease it.

Choosing viewing modes

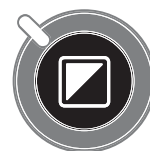Pressing this button will cycle you through 3 different viewing modes:

Please note: If the default grey scale setting is enabled on your ClearView⁺, this button will also include a fourth viewing mode which shows the image in grey scale (true black and white).

1-Photo mode full color (or grey scale with a monochrome unit)

2-Reading mode: This mode enhances the foreground / background contrast. Pictures and text will be displayed in high contrast black and white.

If an optional Feature Pack was purchased with the ClearView⁺, the background and foreground color combination can be changed.

3-Reverse reading mode: This mode reverses the chosen background and foreground color combinations of the reading mode.

4-Photo mode grey scale (if enabled)

Auto-focus lock

The ClearView⁺ features an automatic focus system that produces a sharp on-screen image at any time.

When switching the ClearView⁺ on, the system will start up in the auto-focus mode. Pressing the auto-focus button stops the ClearView⁺ from continuously operating the auto-focus feature and fixes the focus on the object being viewed. This can be useful for tasks, such as writing, painting or crafts. When the auto-focus feature is in locked mode, a red light on the central control button (mode / zoom) will light up. To turn the auto-focus back on, simply

press the auto-focus button again. The red indicator light will turn off.

Object lighting ON /OFF

Pressing this button for four seconds turns off the object lighting. You can choose this option to make it easier to read glossy articles such as magazines and photos. Pressing the button once will advance the viewing mode by one position and will turn the lights back on; or pressing the button for four seconds will turn the object lighting back to the same viewing mode.

Adjusting image control

Press and hold the right or left side of this button to adjust the image control. The image control may be adjusted to make the characters or other on-screen elements sharper, if they are not clearly visible.

Please note: If the default control setting is changed on your ClearView⁺, pressing this button will adjust the brightness.

Adjusting the brightness

Press the left or right adjustment button while pressing the auto-focus button to adjust the brightness.

Please note: If the default control setting is changed on your ClearView⁺, pressing these buttons will adjust the image control.

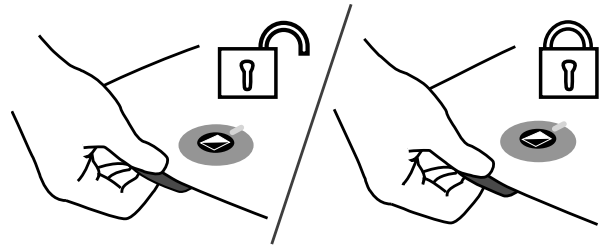
Displaying system information on-screen

The magnification factor and the system configuration (system modules) can be displayed on-screen. Press the auto-focus button for four seconds to have this information displayed. Press it once again for four seconds to turn this feature off.

Locking the reading table

Lift the red lever located at the front of the table to lock the table in a stable position. This is very useful when writing, painting and doing hobbies and crafts. Unlock the table by simply lifting the lever once again.

Please note: When the ClearView⁺ power is off, the brake will automatically lock into position. When the ClearView⁺ is switched on, the brake will unlock. Make sure to push the reading table backwards to its home position prior to turning off.

Using the footswitch

The optional footswitch on the PC unit allows you to toggle between the image of the ClearView⁺ and a connected PC (computer) or other connected VGA source, such as an external camera as well as providing Picture-in-Picture. The footswitch on a Monochrome, Color or HD unit allows you to toggle between the images of the ClearView⁺ and of an external source such as a PC (computer).

Please note: If your ClearView⁺ Color unit is equipped with a 22" TFT monitor, the footswitch allows you to toggle between the ClearView⁺ image, vertical split-screen (showing both the ClearView⁺ image and computer image in the ratio 50:50) and full screen computer image.

5. Feature Packs

Feature Packs provide increased functionality to the ClearView⁺. Optelec offer four Feature Packs: Advanced, Basic, Auto Zoom and Zero Button.

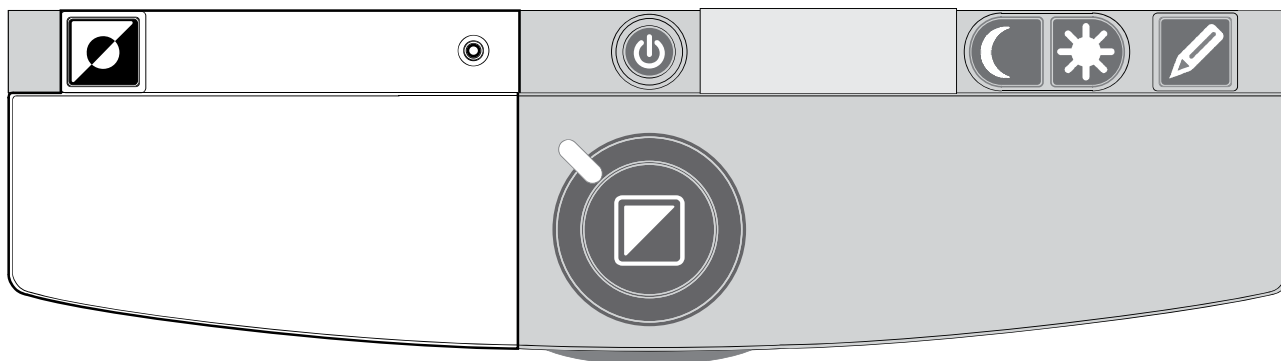
The Feature Pack Basic provides increased functionality to the monochrome, color and color HD units.

The Feature Pack Advanced can be used with all ClearView⁺ units. For PC units, the Feature Pack Advanced is fitted as standard. If you are using the ClearView⁺ PC unit, please refer to chapter 6 PC units with Advanced Feature Pack .

The Zero Button and Auto Zoom Feature Packs offer the unique fixed font functionality on the Monochrome and Color units.

5.1 Feature Pack Basic

The Feature Pack Basic offers selectable semi-color and the position locator functions.

Semi-colors

Pressing this button in reading mode or reverse reading mode cycles you through 7 different foreground and background color combinations. You may choose to display text in a choice of color combinations when additional contrast is needed, or when a different color combination provides increased visibility. Press and hold the button for

two seconds to instantly return to black and white. Press and hold the button once again for two seconds to return to your last selected semi-color combination.

Press this button in photo mode to toggle between the full color and the grey scale modes.

Please note: This function is unavailable on the monochrome unit. If the default grey scale setting is changed on the ClearView⁺, this button will not work in photo mode. In this case, you may choose the grey scale mode by pressing the viewing mode button.

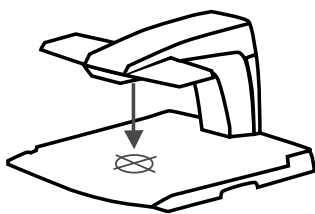
Find function (position locator)

The find function provides you with a quick overview of the text, which helps you easily locate the part of a document which is of interest.

Press the position locator button to activate the find function. When the find function is activated, the ClearView⁺ will automatically zoom out to the minimum magnification, and will display a crosshair target to indicate the centre of the screen. Position the text you wish to read in the crosshair's target by moving the reading table. Once the document has been positioned, press the button once again to zoom in on the selected text in your previous magnification setting.

Pointer (position locator)

Pressing this button for three seconds will activate a red LED light, which shows as a red spotlight on the object in the centre of the reading table. It serves to indicate the centre of the on-screen image. To assist with writing, place a pen in the red spotlight on the reading table and see the pen displayed in the centre of your screen. Press this button once again to deactivate the LED light.

5.2 Feature Pack Advanced

The Feature Pack Advanced contains all the features of the Feature Pack Basic and adds the line marker, window shade and PC-switch functions. For a description of the semi-colors and position locator functions, please refer to section 5.1 Feature Pack Basic.

Switching between the ClearView⁺ and a computer / external source image

If a computer or another external source is connected to the ClearView⁺, press this button or the footswitch if fitted to toggle between the ClearView⁺ image and the full screen computer image.

To display the computer image on your ClearView⁺ monitor correctly, make sure that your computer's resolution is set accordingly:

- For the ClearView⁺ 17" TFT and 19" TFT monitors, the resolution must be set to 1280 x 1024.
- For the ClearView⁺ 18" TFT monitor, the resolution must be set to 1366 x 768.
- For the ClearView⁺ 22" TFT monitor, the resolution must be set to 1680 x 1050.

Please note: If your ClearView⁺ color unit is equipped with a 22" TFT monitor, pressing this button will allow you to toggle between the ClearView⁺ image, vertical split-screen (showing both the ClearView⁺ image and

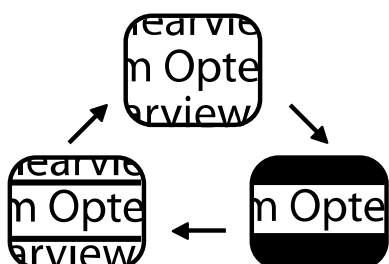
computer image in the ratio 50:50) and full screen computer image. The split-screen image may not be resized or repositioned.

Using ClearView⁺ monitors and the ClearView⁺ color HD unit do not provide the split-screen image function.

Activating line markers / window shades



Pressing this button cycles you through the line markers, window shades and full screen camera image modes. Line markers are used as guides to make reading text easier. The window shades function may be used to block a part of an image when screen brightness becomes inconvenient or when needing to concentrate on only one or two lines of text is preferred.

Please note: Line markers and window shades may only be used in reading mode or in reverse reading mode. This option is unavailable in photo color or grey scale mode, or with the vertical split-screen of the ClearView⁺ color unit.

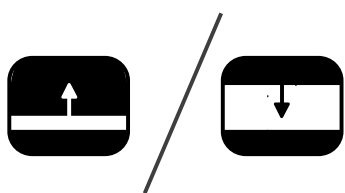
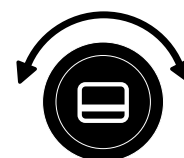
Sizing the lines markers / windows

When line markers or window shades are activated, turning this button will narrow or widen the space between the line markers / window edges.

This can either be modified horizontally or vertically by using this button

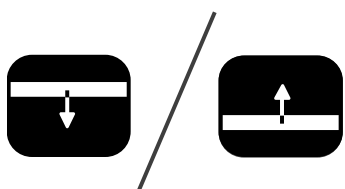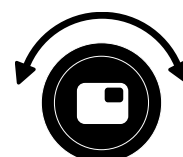
Positioning lines markers / windows shades

When line markers or window shades are activated, turning this button will change their position on screen.

Line markers and window shades may be displayed horizontally and vertically depending upon the status of this button described below .

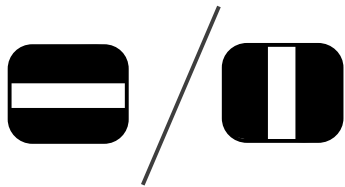
Horizontal / vertical button

Pressing this button allows you to  switch the line markers / windows from the vertical to the horizontal position.

5.3 Zero Button Feature Pack

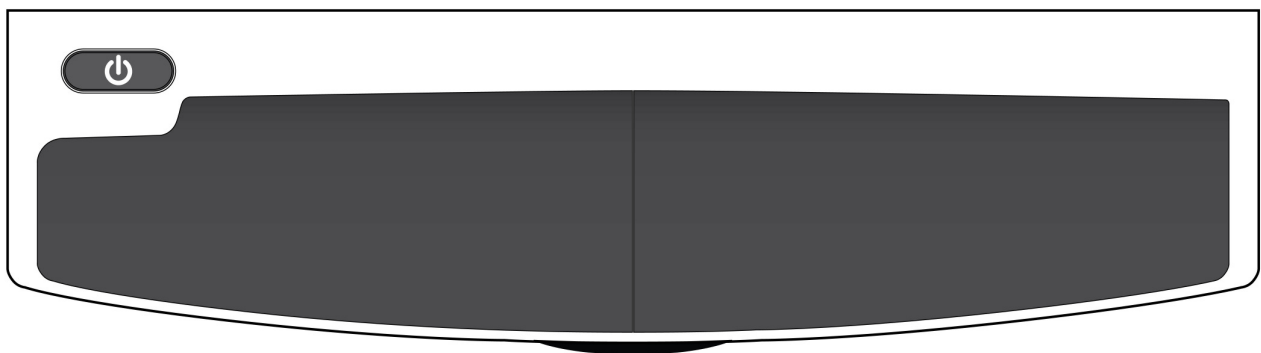The Zero Button Feature Pack makes reading effortless thanks to the unique Auto Zoom fixed font technology.

Switching ON / OFF

Press the On / Off button located on the left side, above the wrist pad, to turn both the system and the monitor on. A non-ClearView⁺ monitor must be switched On / Off separately.

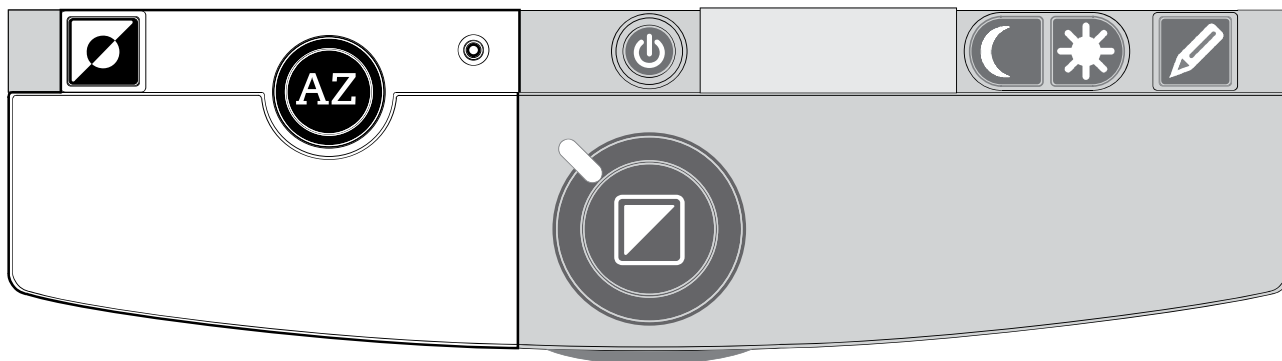
Auto Zoom fixed font technology

The Zero Button Feature Pack features the unique Auto Zoom fixed font technology. This feature automatically converts and displays the text that is placed on the reading table in your preferred fixed font size and viewing mode.

For more information about the Zero Button Feature Pack, please refer to the separate Zero Button User Manual.

5.4 Auto Zoom Feature Pack

The Auto Zoom Feature Pack features the same functions as the Feature Pack Basic and the unique Auto Zoom fixed font technology. For a description of the semi-color and position locator functions, please refer to section 5.1 “Feature Pack Basic”.

Auto Zoom ON/OFF

Press this button to activate the Auto Zoom function.

The Auto Zoom function will automatically display the text that is placed on the reading table in your preferred fixed font size regardless of the original document font size .

When the Auto Zoom function is activated, the zoom ring located in the centre of the reading table will be disabled. Such functions as viewing mode, semi-colors, position locator and brightness will still be available.

Press the Auto Zoom button once again to return to the manual zoom mode.

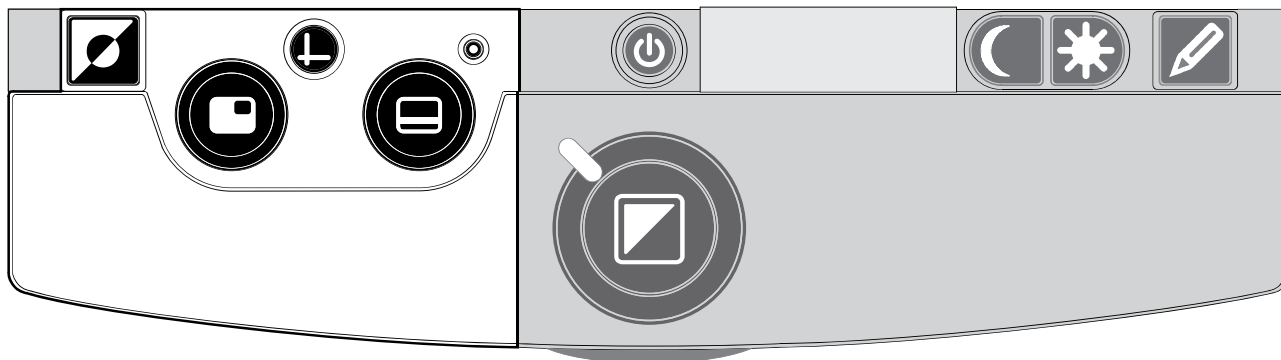
Adjusting the Auto Zoom

The fixed font size function of the Auto Zoom feature may be adjusted by turning the Auto Zoom button. Turn the button clockwise to increase the fixed font size and counter-clockwise to decrease it.

6. PC units with the Feature Pack Advanced

The Feature Pack Advanced is fitted as standard on the PC units.

Semi-color combinations

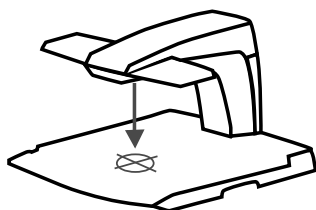
Pressing this button in high contrast reading modes cycles you through 7 different foreground and background color combinations. You may choose to display text in a selection of color combinations when additional contrast is needed, or when a different set of color combination provides increased visibility. Press and hold the button for two seconds to instantly return to black / white.

Pointer (position locator)


Pressing this button will activate a red LED light, which shows as a red spotlight on the object in the centre of the reading table. It serves to indicate the centre of the on-screen image. To assist with writing place a pen in the red spotlight on the reading table and see the pen displayed in the centre of your screen. Press this button once again to deactivate the LED light.

Activating line markers / window shades

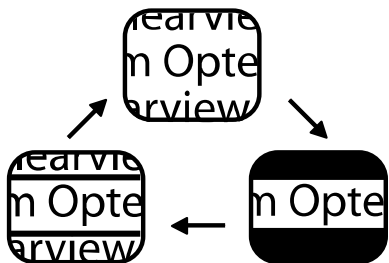Pressing this button cycles you through line markers, window shades and full-screen camera image. Line markers are used as guides to make reading text easier. The window


shades function may be used to block a part of an image, when the screen brightness becomes inconvenient or when needing to concentrate on only one or two lines of text is preferred.

Please note: Line markers and window shades may only be used in reading mode or in reverse reading mode. This option is unavailable in photo color or grey scale mode, or with the vertical split-screen of the ClearView⁺ color unit.

Switching between the ClearView⁺ and a computer / external source image

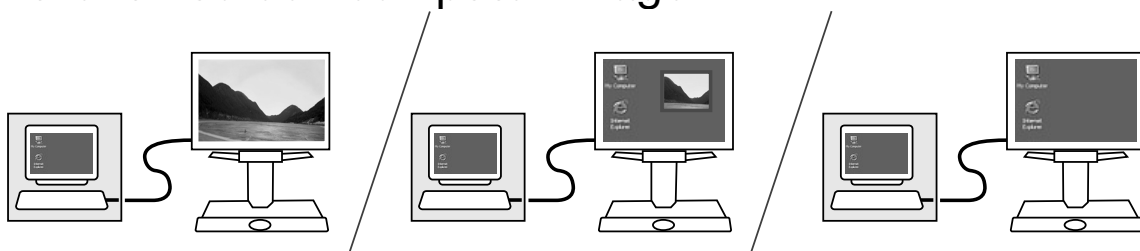If a computer or another external source is connected to the ClearView⁺, press this button or the footswitch, if fitted, to toggle between the ClearView⁺ image, Picture-in-Picture (ClearView⁺ image as a window within the computer image) or the full-screen computer image.



Sizing the line markers / window shades & Picture-in-Picture image



When line markers or window shades are activated, turning this button will allow you to modify the space between the line markers / window edges in either in a

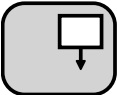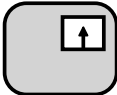horizontal or vertical position depending on the status of the horizontal / vertical button described below .



 /  When Picture-in-Picture is activated, turning this button will allow you to change the size of the Picture-in-Picture image, depending on the status of the horizontal / vertical button described below .

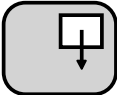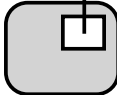Positioning line markers / window shades & the Picture-in-Picture image

 /  When line markers, window shades or the Picture-in-Picture are activated, turning this button will change the position of the line markers / window shades or Picture-in-Picture image. Positioning can either be modified horizontally or vertically by pressing this button .

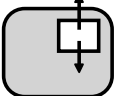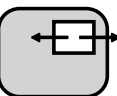Horizontal / vertical button

 /  Pressing this button allows you to switch the line markers and window shades from a vertical to a horizontal position. 

7. Adjusting the monitor

The special ClearView⁺ 17" and 18" TFT monitors can be adjusted in height and viewing angle for an optimal reading position. Adjusting the monitor assists in preventing glare. Firmly grip the flat panel on both sides to adjust to your preferred position.

The unique ClearView⁺ 19" and 22" TFT monitors are delivered with a five-directional ultra-flexible TFT arm which allows you to adjust the monitor to a preferred position. The monitor can be moved up or down, forward or backward, and tilted horizontally and vertically for optimal ergonomic and viewing comfort. The rotating feature enables you to turn the monitor clockwise 90° making column reading possible. When rotating the monitor, the image will automatically switch to portrait viewing mode. The automatic image rotation can be enabled or disabled by your Optelec distributor.

To place the monitor in your preferred position, firmly grip the flat panel on both sides by the surrounding frame, and move, tilt or rotate it as required. You will feel a firm click which will indicate the centre tilt and the maximum end positions of the rotation.

Please refer to the poster included in the monitor box for more information about adjusting the ultra-flexible TFT arm.

Avoid putting any magnetic or metal objects in front of or near the monitor as they may damage the monitor.

8. Troubleshooting

Manufacturer's default settings

If you experience any difficulty operating the system, you can reset the factory default settings. Switch the ClearView⁺ off and wait 3 seconds. Then press and hold the On / Off button till the red ring light starts flashing.

No image on the ClearView⁺ monitor after it is switched on

- Check all cables and hardware are connected properly and securely.
- Check that the unit is receiving power by verifying that the green indicator LED light is lit on the power supply.
- Ensure sure the monitor is switched on if a non-ClearView⁺ monitor is connected.

Image is not in focus

- Check the auto-focus lock (the lit red circle around the mode button means that the focus is locked). Press the focus button to return to auto-focus mode.
- Carefully clean the mirror on the camera unit with a non-abrasive cloth.
- Clean the monitor screen with a non-abrasive cloth.

Pictures and text look grey or have low contrast

- Press the mode button to switch to the high contrast reading mode.
- Adjust the image control or brightness level on the control panel.
- If you are using a non-ClearView⁺ monitor, modify the monitor's brightness and contrast settings .

The computer image is not displaying

- Check to see if the Advanced Feature Pack is installed.
- Verify the computer's video (VGA) connection to the base unit (do not connect the PC directly to the monitor).
- Check the status of the  button.
- When operating with the footswitch, check to see if the Advanced Feature Pack is installed.
- Check the computer for any malfunction by connecting the PC directly into the monitor.

Picture-in-Picture mode does not work

- Ensure that your system is a PC system.
- Ensure that the computer video resolution is set to the right resolution. See page 17 of this manual.

Picture jerks in Picture-in-Picture mode (PC units only)

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
- Ensure that the computer video resolution is set to the right resolution. See page 17 of this manual.

The reading table does not move or does not move smoothly

- Unlock the table by pulling the lever, located under the front of the table.
- Inspect the tracks for dust or any other obstructions.

9. Contacting Product Support

If you have questions about operating, installing or setting your ClearView⁺, please contact your Optelec distributor or the Optelec office nearest you. See the last pages of this manual for a complete list of Optelec addresses.

10. Warranty Conditions

Optelec warrants the ClearView⁺, effective from the date of delivery, to be free of any defects in material and workmanship.

The warranty is not transferable and does not apply to groups, multi-users and agencies. The ClearView⁺ was designed for the individual purchaser to be used at home and work. Optelec reserves the right to repair or replace any purchased ClearView⁺ with a similar or improved product.

In no event shall Optelec or its suppliers be held liable for any indirect or consequential damages. The original user's remedies are limited to replacement of ClearView⁺ modules. This warranty is valid only when serviced in the country of original purchase, and with intact seals. For additional warranty claims or service during or after warranty period, please contact your Optelec distributor.

Optelec is not responsible for any uses of this device other than those described in this manual.

11. Compliance

This product complies with the safety and EMC requirements under the following directives:

MDD:

EN 12182

EN 60601-1-2 (EMC)

EN 60601-1 (safety)

FCC

FCC part 15 Class B

C-Tick

CISPR 22

EMC:

EN 50081-1

EN 55022

EN 55024

Safety

UL 60950

Electrical medical equipment requires special precaution with regard to EMC (Electro Magnetic Compatibility) and must be installed and put into service according to EMC guidelines.

WARNING: The use of accessories and cables other than those specified and sold by the manufacturer of the ClearView⁺, as replacement parts for components, may result in increased emissions or decreased functioning of the ClearView⁺.

This device bears the CE, FCC and UL approval mark.

12. Specifications

Focus	: Auto-focus with focus lock
Magnification – Monochrome, Color and PC units	: 2.6 X – 50 X on a 17" screen
Magnification – color HD units	: 1 X – 50 X on a 17" screen
Dimensions base unit footprint	: 44 x 50 cm
Power supply	: 110 - 240 Volt AC. – 12 Volt / 8.3A max.
Net frequency	: 50 - 60 Hz

The ClearView⁺ PC unit supports:

Resolution	Monitor output
640 x 480	60 Hz
800 x 600	60 Hz
1024 x 768	60 Hz
1280 x 1024	60 Hz

Operating conditions

Temperature	: +10°C to +35°C
Humidity	: < 70%, no condensation

Storage & transportation conditions

Temperature	: -10°C to +40°C
Humidity	: < 95%, no condensation

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Optelec GmbH
Optelec US Ltd
U.S.A.