Do not print this page!

Specifications

The user manual is A5 landscape format.

The covers need to be one-side color-printed on A4 paper 200 gram (PRT-00245) separately!

- Front cover
- Back cover

The content pages need to be:

- black & white
- double-side printed
- on A4 paper 80 gram (PRT-00244)!
- starting from the first page of the English language manual (English - Page 1)

The following content pages need to be color printed:

First pages of the different languages (with Optelec logo)
Configuration pages in the appendix

Paper binder (PRT-00246): A5 length 297 mm

For the Binding instruction please refer to RFD-00321

ClearView⁺

User Manual
Gebruikershandleiding
Manuel d'utilisation
Manuale Utente
Bedienungsanleitung

Zero Button Feature Pack

ClearView⁺ Zero Button Feature Pack

User Manual

OPN: CV-FP-ZB, Version 1
© 2008 Optelec, the Netherlands
All rights reserved

P.O. Box 399
2990 AJ Barendrecht
The Netherlands
Phone: +31 (0)88 678 34 44
E-mail: info@optelec.nl
Internet: www.optelec.com

Contents

1.	Introduction.....	3
	About this manual.....	3
2.	Getting to know your Zero Button Feature Pack.....	4
3.	Getting started.....	4
	Switching On / Off.....	5
	AutoZoom functionality	5
	Reading documents.....	5
	Viewing photos	6
	Locking the reading table	6
4.	Adjusting the reading settings	7
	Opening the ClearView ⁺ configuration menu	7
	Menu buttons.....	8
	AutoZoom.....	9
	Mode	9
	Brightness	10
	Image control.....	10
	Restore defaults	11
	About	11
5.	Using configuration pages	12

1. Introduction

Congratulations on choosing the ClearView⁺ with Zero Button Feature Pack, a desktop video magnifier system with unique AutoZoom functionality designed by Optelec.

The Zero Button Feature Pack features an enhanced design without operating buttons that extends in reading comfort. Regardless of the document's font size, the integrated AutoZoom functionality automatically converts and displays printed information in the preferred magnification and viewing mode. It is designed for everyday use at home or in an educational or office setting.

If you have any questions or suggestions concerning the use of this product, please contact your distributor or the Optelec headquarters using the contact information at the last page of the ClearView⁺ manual. Your feedback is greatly appreciated. We hope you enjoy working with your ClearView⁺ with Zero Button Feature Pack.


About this manual

This manual will familiarize you with the features and basic operation of the Zero Button Feature Pack. Please read this manual carefully before using your device.

2. Getting to know your Zero Button Feature Pack

The Zero Button Feature Pack is a module of the ClearView⁺. When installed, it is integrated into the reading table.

The following figure shows the ClearView⁺ with the Zero Button Feature Pack.

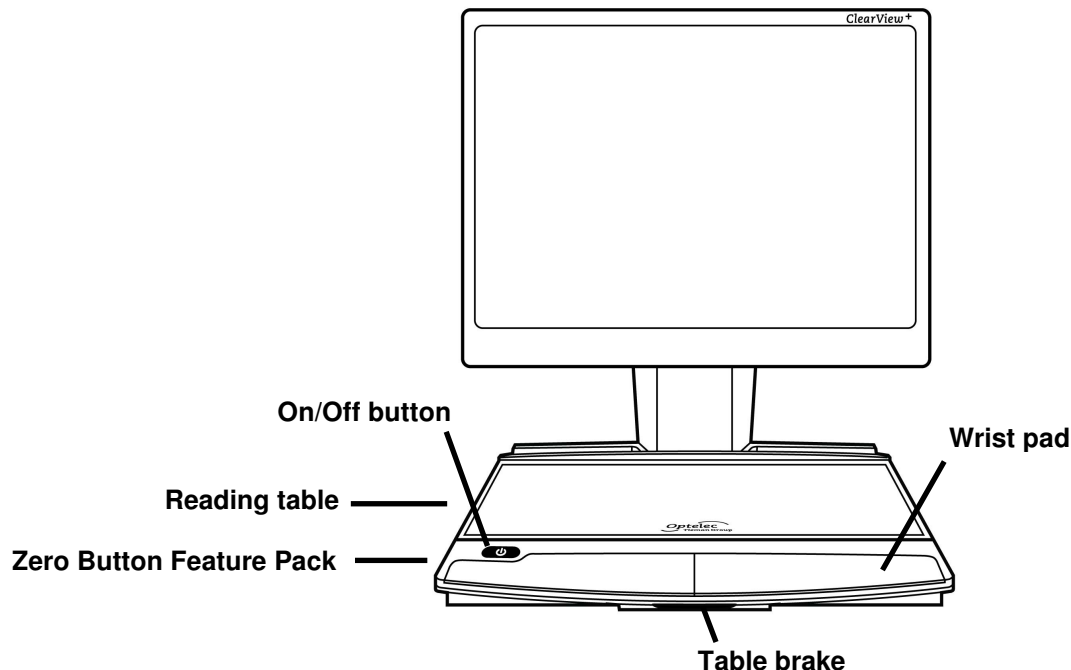3. Getting started

The ClearView⁺ will start using its default settings the first time it is switched on. For information on how to change the settings please refer to chapter 4.

Switching On / Off

Press the orange On / Off button located on the left side of the Zero Button Feature Pack above the wrist pad to power both the system and the monitor. A non-ClearView⁺ monitor needs to be switched On / Off separately.

AutoZoom functionality

The AutoZoom functionality automatically converts and displays the text placed on the reading table in your preferred pre-set magnification and viewing mode.

Reading documents

Place a document on the reading table and make sure the information you wish to read is visible in the centre part of the monitor. The AutoZoom function will automatically adjust the text to your preferred pre-set magnification and reading mode. Move the table left and right, up and down to read the document. If the text contains a different font size, place the line with the smaller or larger font size in the centre of the screen. After approximately 2 seconds, the AutoZoom functionality will


automatically adjust to the pre-set magnification. While reading, rest your arms on the comfortable wrist pads for optimal reading comfort.

Viewing photos

The Zero Button Feature Pack features automatic photo detection enabling you to view photos and images. The ClearView⁺ will automatically switch to full color photo mode when a picture or image has been detected. When placing text under the camera, the AutoZoom functionality will automatically switch back to the pre-set reading settings. Continuously moving the table after a photo has been detected enables you to view the complete document in full color photo mode in minimum magnification.

Locking the reading table

To lock the reading table locate the red lever on the underside of the Zero Button Feature Pack at the front of the table as illustrated in the figure. Press the red lever upwards to lock the table movement. Unlock the table by pressing the lever upwards again.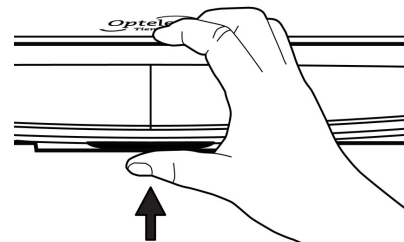


Note: When the ClearView⁺ is switched off, the reading table will automatically be locked. Before switching off the unit make sure to position the reading table in its “home” position. The reading table stands in the “home” position when it is pushed against the backside of the unit aligning with the sides.

4. Adjusting the reading settings


The default reading settings can be adjusted via the configuration menu or by using the configuration pages at the end of this manual. To change the reading settings place a text document on the reading table of the ClearView⁺. While configuring the settings using the menu the adjustments will be immediately visible on the screen. For information on how to use the configuration pages, please refer to chapter 5.

Opening the ClearView⁺ configuration menu


To open the ClearView⁺ configuration menu please remove the left wrist pad of the Zero Button Feature Pack: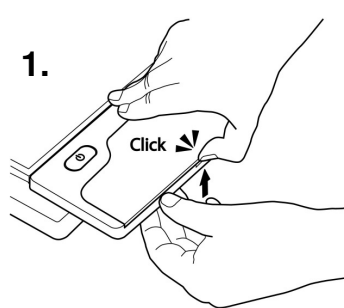

Lift up the left part of the Zero Button wrist pad until you hear a click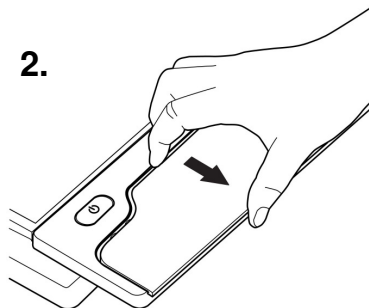


Pull the left wrist pad towards you

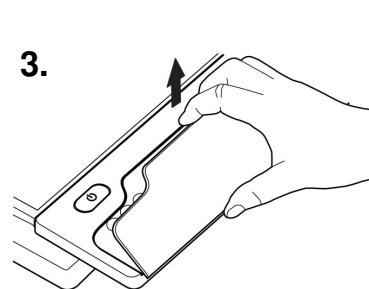Remove the left wrist pad

The ClearView⁺ configuration menu has 7 items. These allow you to configure the reading settings. The ClearView⁺ configuration menu can be operated by three navigation buttons located under the left wrist pad. To enter the configuration menu press the middle button. The ClearView⁺ configuration menu has the following settings:

- AutoZoom
- Mode
- Brightness
- Image control (only available if “Full color photo” mode is selected as pre-set reading mode)
- Restore defaults
- About
- Exit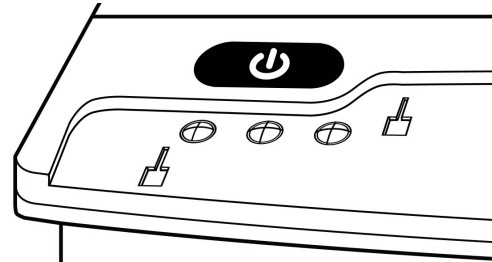

Menu buttons

To enter the ClearView⁺ configuration menu press the middle button. Pressing the left and right buttons allow you to navigate respectively to the previous or next menu item. Press the middle button to select a setting. When a setting is selected the value can be adjusted using the left or right button. All changes will immediately be visible on the screen. Once the preferred values have been selected, press the middle button to confirm the settings. To exit the configuration menu select “Exit”.

AutoZoom

The AutoZoom setting can be adjusted according to points (pt), a unit for measuring font sizes. The AutoZoom can be set between “35 pt” and “350 pt”. The default value is 100 pt, translating to approximately 6 lines of text being displayed on the screen. If you prefer a larger magnification, select a value above 100 pt. If you prefer a smaller magnification, select a value below 100. For more detailed information about the different AutoZoom values, please refer to the text on the configuration pages at the end of this manual.

Note: For optimal reading comfort we recommend setting the AutoZoom value one lever higher than your preferred magnification level.

Mode

The default reading mode is the negative mode “Inverse reading mode” (white text on black background).

The Zero Button Feature Pack features the following viewing modes:

- Photo mode (full color, greyscale): view pictures and images in full color or greyscale)
- Positive mode: reading in high contrast black on white.
- Negative mode: reading in high contrast white on black.

- High contrast semi color modes: reading in 14 different high contrast color modes (e.g. Blue on Yellow).

The following reading modes can be selected in the configuration menu:

- | | | |
|--------------------|-------------------|-------------------|
| – Full color photo | – Black on Orange | – Yellow on Blue |
| – Black on White | – Black on Red | – White on Blue |
| – Black on Yellow | – Black on Purple | – Orange on Black |
| – Black on Green | – White on Black | – Red on Black |
| – Blue on Yellow | – Yellow on Black | – Purple on Black |
| – Blue on White | – Green on Black | – Greyscale photo |

Brightness

Select the level of brightness of the monitor. If you prefer maximum brightness set the value to 5. If you prefer a low level of brightness, set the value to 1. The default brightness level is 5.

Image control

Image control will be available as a menu items if “Full color photo” is selected as a pre-set reading mode. If you prefer less contrast, decrease the value to 1. If you prefer more contrast, increase the level to 5. The default value is 4.

Restore defaults

For restoring the default reading settings select “Restore defaults” in the menu and choose “Yes”. Confirming will restore the settings to:

AutoZoom: 100 pt

Mode: White on Black

Brightness: 5

Image control: 4 (only visible if the Full color photo mode is selected)

About

Select “About” in the ClearView⁺ configuration menu to display information about the ClearView⁺ unit such as model code, hardware number, firmware version and firmware revision.


5. Using configuration pages

The configuration pages contain codes to adjust the settings of your ClearView⁺.

Please refer to the configuration pages in the appendix at the end of this manual. The configuration pages are grouped according to the settings: AutoZoom, mode and brightness. Above the codes you will find the value of the setting (e.g. AutoZoom value “35 pt”). For a detailed description of the settings please refer to chapter 4.

Adjust the settings with the configuration pages as follows:

1. Switch on the ClearView⁺
2. Ensure that the reading table is locked and moved to the “home position”.
3. Place the configuration page on the reading table and position the selected barcode in the middle of the screen as the figure shows.
4. Wait approximately 2 seconds while the unit adjusts to the new settings and the changes are displayed on the screen.

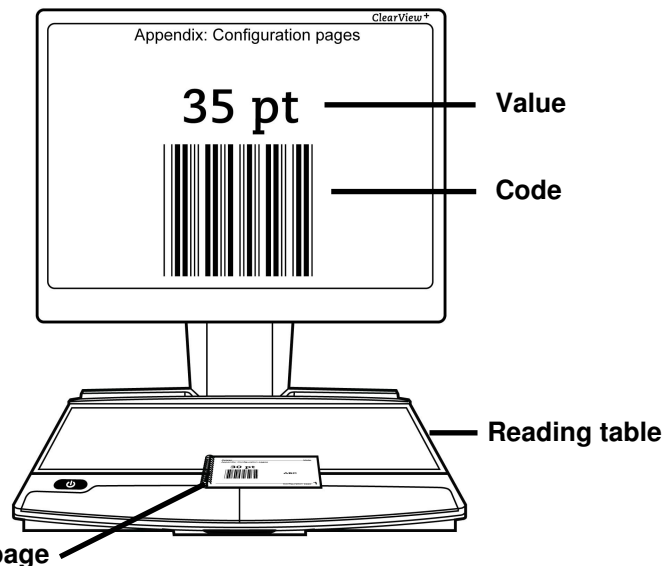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Optelec Limited
Tieman GmbH